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Kontakt: www.menoelle.com hello@menoelle.com

English	Ingredients
<i>Please read these instructions carefully before use.</i>	Water, linseed extract*, aloe vera*, locust bean gum*, guar gum*, sodium chloride, xanthan gum, potassium sorbate, citric acid, phenoxyethanol.
menoelle Gel	<i>*Certified organic ingredients. 95 % organic.</i>
menoelle Gel is a certified organic moisturising gel for vaginal use. menoelle Gel uses patented technology and combats vaginal dryness with natural, long-lasting rehydration.	International patent protection.
<ul style="list-style-type: none"> When oestrogen levels drop, e.g. during menopause, this can lead to vaginal irritation, itching and burning sensations. menoelle Gel provides effective, swift and long-lasting relief and ensures physical comfort. menoelle Gel uses bio-adhesive plant polymer that binds to the vaginal walls and provides hydration for up to three days. Case studies have shown that menoelle Gel achieves a fast and long-lasting result and hydrates the vagina in a safe and natural way. 	International patent protection.
Use	Use
menoelle Gel is a gentle, colourless gel that can be simply applied and distributed with your finger tips (no applicator supplied). Therefore, you can decide yourself, how much and where you wish to apply it. Squeeze the desired amount of gel from the tube and distribute it with your fingertips to the inside walls of your vagina. menoelle Gel can also be used to moisten the lips of the vagina. Product does not come with a reusable applicator.	menoelle Gel is a gentle, colourless gel that can be simply applied and distributed with your finger tips (no applicator supplied). Therefore, you can decide yourself, how much and where you wish to apply it. Squeeze the desired amount of gel from the tube and distribute it with your fingertips to the inside walls of your vagina. menoelle Gel can also be used to moisten the lips of the vagina. Product does not come with a reusable applicator.
Frequency of use	Frequency of use
Application of 5 ml every two to three days is usually enough to keep vaginal tissues hydrated and comfortable. In cases of extreme dryness, the gel can initially be used more frequently (1x every one to two days), until there is an improvement. For easing pain during sexual intercourse, the gel should be applied before intercourse and again afterwards to provide relief.	Application of 5 ml every two to three days is usually enough to keep vaginal tissues hydrated and comfortable. In cases of extreme dryness, the gel can initially be used more frequently (1x every one to two days), until there is an improvement. For easing pain during sexual intercourse, the gel should be applied before intercourse and again afterwards to provide relief.
Notice	Notice
menoelle Gel:	isotonisch* : The isotonic/iso-osmotic property of menoelle Gel is the same as that of human tissue. Why is this balance necessary? Hypertonic products (which generally contain chemicals like glycerine and glycol) can cause vaginal irritation. menoelle Gel 's isotonic/iso-osmotic property protects against irritation.
<ul style="list-style-type: none"> a gentle, clear and natural bio-adhesive gel discreet, non-sticky, colourless, fragrance and taste free. Mimics the consistency and feel of the body's natural vaginal lubrication the correct pH for restoring and maintaining the protective acidity level in the vagina, fosters a healthy vaginal flora is not made with hormones not made with any substances known to irritate mucous membranes, and free from ingredients such as glycerine, oil and petroleum-based products, parabens (hormone mimics) that may alter or damage the healthy vaginal flora. gentle and natural, suitable even in cases of extreme sensitivity guaranteed pure with certified organic status safe for use with condoms made of latex and polyisoprene proven by health experts and by women to provide soothing, safe and pleasant relief from the uncomfortable symptoms caused by vaginal dryness during menopause a hormone-free, natural alternative to synthetic and chemical vaginal rehydration gels. 	isotonisch* : The isotonic/iso-osmotic property of menoelle Gel is the same as that of human tissue. Why is this balance necessary? Hypertonic products (which generally contain chemicals like glycerine and glycol) can cause vaginal irritation. menoelle Gel 's isotonic/iso-osmotic property protects against irritation.
Storage: When not in use, always store the tube with the lid closed. Store menoelle Gel in a cool, dry place at temperatures of between +5 and +30 °C. Protect against direct sunlight. Use before expiry date. Caution: menoelle Gel is neither a spermicide nor a contraceptive.	Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice.
<i>Not compatible with polyurethane condoms.</i>	Not compatible with polyurethane condoms.
Certified organic by: The Soil Association, Bristol, UK.	Certified organic by: The Soil Association, Bristol, UK.
Approved by: The Vegetarian Society.	Approved by: The Vegetarian Society.
Distributor	Distributor
Distributor and import: PhytoLife Nutrition GmbH Winsstrasse 59 10405 Berlin Germany	Distributor and import: PhytoLife Nutrition GmbH Winsstrasse 59 10405 Berlin Germany
Manufacturer: The YES YES Company Ltd. The Timber Barn, Selborne Road Greatham, Liss, Hampshire GU33 6HG UK	Manufacturer: The YES YES Company Ltd. The Timber Barn, Selborne Road Greatham, Liss, Hampshire GU33 6HG UK

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Distribuidor e importador: PhytoLife Nutrition GmbH Winsstrasse 59 10405 Berlín Alemania
Fabricante: The YES YES Company Ltd. The Timber Barn, Selborne Road Greatham, Liss, Hampshire GU33 6HG Reino Unido
Satış ve Dağıtım: PhytoLife Nutrition GmbH Winsstrasse 59 10405 Berlin Almanya
Üretici: The YES YES Company Ltd. The Timber Barn, Selborne Road Greatham, Liss, Hampshire GU33 6HG Birleşik Krallık