Français

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English

Please read these instructions carefully before use.

menoelle Gel

menoelle Gel is a certified organic moisturising gel for vaginal use. **menoelle Gel** uses patented technology and combats vaginal dryness with natural, long-lasting rehydration.

- » When oestrogen levels drop, e.g. during menopause, this can lead to vaginal irritation, itching and burning sensations. **menoelle Gel** provides effective, swift and long-lasting relief and ensures physical comfort.
- » **menoelle Gel** uses bio-adhesive plant polymer that binds to the vaginal walls and provides hydration for up to three days.
- » Case studies have shown that **menoelle Gel** achieves a fast and long-lasting result and hydrates the vagina in a safe and natural way.

provides symptomatic relief for:

- » Vaginal dryness
- » Menopausal vaginal atrophy
- » Itchiness
- » Irritation, burning sensations and sensitivity
- » Pain during sexual intercourse

Temporary vaginal dryness can have various causes: Birth, breastfeeding, taking the pill, or hormonal changes during menopause. Certain antidepressant and antihistamine medication, as well as extreme stress, can also lead to vaginal dryness.

menoelle Gel:

- » a gentle, clear and natural bio-adhesive gel
- » discreet, non-sticky, colourless, fragrance and taste free. Mimics the consistency and feel of the body's natural vaginal lubrication
- » the correct pH for restoring and maintaining the protective acidity level in the vagina, fosters a healthy vaginal flora
- » is not made with hormones
- » not made with any substances known to irritate mucous membranes, and free from ingredients such as glycerine, oil and petroleum-based products, parabens (hormone mimics) that may alter or damage the healthy vaginal flora.
- » gentle and natural, suitable even in cases of extreme sensitivity
- » guaranteed pure with certified organic status
- » safe for use with condoms made of latex and polyisoprene
- » proven by health experts and by women to provide soothing, safe and pleasant relief from the uncomfortable symptoms caused by vaginal dryness during menopause
- » a hormone-free, natural alternative to synthetic and chemical vaginal rehydration gels.

Isotonisch*: The isotonic/iso-osmotic property of **menoelle Gel** is the same as that of human tissue. Why is this balance necessary? Hypertonic products (which generally contain chemicals like glycerine and glycol) can cause vaginal irritation. **menoelle Gel**'s isotonic/iso-osmotic property protects against irritation.

**Further information can be found on our website.*

Storage: When not in use, always store the tube with the lid closed. Store **menoelle Gel** in a cool, dry place at temperatures of between +5 and +30 °C. Protect against direct sunlight. Use before expiry date or within 4 months of opening.

Caution: menoelle Gel is neither a spermicide nor a contraceptive.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice

*Not compatible with polyurethane condoms.
Not suitable for use with the ThinPrep Pap Test.*

Certified organic by: The Soil Association, Bristol, UK.
Approved by: The Vegetarian Society.

sation/allergic reaction may occur.

Report Serious side effects to PhytoLife Nutrition GmbH and your local health authority

Ingredients

Aqua (water)¹, Linum usitatissimum (flax) extract^{2,3}, Aloe barbadensis leaf juice powder (aloe vera)^{2,3}, Ceratonia siliqua (locust bean gum)^{2,3}, Cyamopsis tetragonolobus (guar gum)^{2,3}, Sodium chloride², Xanthan gum², Potassium sorbate, Citric acid, Phenoxyethanol.

²Certified organically grown ingredients (95%)
³Ingredients responsible for product effectiveness (>99%)

International patent protection.

Use
menoelle Gel is a gentle, colourless gel that can be simply applied and distributed with your finger tips (no applicator supplied). Therefore, you can decide yourself, how much and where you wish to apply it. Squeeze the desired amount of gel from the tube and distribute it with your fingertips to the inside walls of your vagina.

menoelle Gel can also be used to moisten the lips of the vagina. **Product does not come with a reusable applicator.**

menoelle Gel is intended for use by adults and is not intended for use on broken or damaged skin

Frequency of use

Application of 5 ml every two to three days is usually enough to keep vaginal tissues hydrated and comfortable.
In cases of extreme dryness, the gel can initially be used more frequently (1x every one to two days), until there is an improvement.
For easing pain during sexual intercourse, the gel should be applied before intercourse and again afterwards to provide relief.

Interactions with vaginally applied medications

menoelle Gel can be safely used with vaginal dilators/trainers/menstrual cups (including those made from silicone) and natural rubber latex or polyisoprene condoms. **menoelle Gel** should not be applied at the same time as other medications such as vaginal HRT, vaginal antifungal or antibacterial medications without consulting your pharmacist/physician.

Notice

Isotonich*: The isotonic/iso-osmotic property of **menoelle Gel** is the same as that of human tissue. Why is this balance necessary? Hypertonic products (which generally contain chemicals like glycerine and glycol) can cause vaginal irritation. **menoelle Gel**'s isotonic/iso-osmotic property protects against irritation.

**Further information can be found on our website.*

Storage: When not in use, always store the tube with the lid closed. Store **menoelle Gel** in a cool, dry place at temperatures of between +5 and +30 °C. Protect against direct sunlight. Use before expiry date or within 4 months of opening.

Caution: menoelle Gel is neither a spermicide nor a contraceptive.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice

*Not compatible with polyurethane condoms.
Not suitable for use with the ThinPrep Pap Test.*

Certified organic by: The Soil Association, Bristol, UK.
Approved by: The Vegetarian Society.

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