menoelle OVULA med + care

wohltuende
Calendula soothing
Calendula

Vaginalzäpfchen | Vaginal Ovules | Óvulos vaginales
| Ovuli vaginali | Ovules vaginaux

DEUTSCH

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ENGLISH

Thank you for purchasing menoelle OVULA med+care!

Please read the following product and application information carefully and keep this leaflet for future reference.

menoelle OVULA med+care is a medical device useful as an adjuvant in the prophylaxis and treatment of bacterial vaginosis, mycotic vaginitis as well as in case of vaginal dryness. Boric Acid and Lactic Acid help eliminate symptoms resulting from an imbalance in vaginal pH, while Hyaluronic Acid promotes the hydration of the vaginal cavity.

Ingredients:

Boric acid, 18 beta glycyrrhetic acid, Hyaluronic acid, Lactic acid, Centella asiatica, Calendula officinalis, Tea tree oil, semi-synthetic triglycerides.

Dosage and method of use

Introduce into the vagina 1 ovule per day for 5-7 days following the menstrual cycle or according to medical indication.

1. Wash your intimate parts thoroughly (preferably with both an external and internal vaginal douche)
2. Open the package and, after washing your hands well, take a single vaginal ovule from the blister
3. Insert the vaginal ovule gently into the vagina trying to reach deep enough to avoid accidental leakage

Storage Conditions:

Store at room temperature away from light and heat sources. Do not use the product after the expiry date. Do not use the product if the packaging is not intact. The expiry date refers to the undamaged and correctly stored product.

Warnings:

Do not ingest. Keep out of sight and reach of children under 3 years of age to avoid accidental ingestion. Do not apply on broken skin. Do not use if you are hypersensitive to one or more of the ingredients. In case of contact with the eyes rinse thoroughly with water.

Precautions of Use:

During pregnancy, lactation the use of ovules must be limited only if strictly required and under direct control of your doctor. Disposable device. In case of accidental exit of the ovule out of the vagina, do not reuse the ovule. Not for the use in children.

Distributor: PhytoLife Pharma GmbH, Winsstr. 59, 10405 Berlin, GERMANY
Manufacturer: PHARMA BIOMATECK srl, Consorzio Fracta Labor L25, 80027 Frattamaggiore (NA), ITALY

€ 10 vaginal ovules of 2g/ovule

The medical device must not be disposed in the wastewaters or household waste. This will help protect the environment.

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Distributore: PhytoLife Pharma GmbH, Winsstr. 59, 10405 Berlin, GERMANY
Distributeur: PhytoLife Pharma GmbH, Winsstr. 59, 10405 Berlin, ALLEMAGNE